

Decision-making support

Introducing our new plan-on-a-page (POP) tool. We've worked in partnership with nurses and specialist woundcare clinicians to develop a unique wound care support tool.

Supporting healthcare professionals for over 150 years

Assess patient, wellbeing and wound

Record type of wound, location, size, state of wound bed, signs of maceration and infection, pain levels, relevant systemic problems, past medical history, concordance













Bring in other healthcare professionals to promote holistic care

Refer appropriately to other professionals for their clinical input: e.g. surgical team, link nurse, tissue viability nurse, dietician, pain team, diabetes team, and podiatry

Control or treat systemic causes

E.g. systemic infection, diabetes, heart/lung conditions, nutritional problems, continence, mobility, vascular issues, pain, stress, anxiety

Decide appropriate treatment

T Tissue non-viable ¹⁻²	I Infection and/or Inflammation ¹⁻²	M Moisture imbalance ¹⁻²	E Edge of wound non advancing ¹⁻²
   	  	   	
A Necrotic B Slough C Granulating unhealthy D Granulating healthy	E Infected F Deep infected cavity wound G Suspected biofilm	H Low H Moderate H High I Depth	J Non-advancing

Identify primary intervention

Hydrogel	Deslougher	Antimicrobial	Foam	Antimicrobial	Antimicrobial	Antimicrobial	Foam or NPWT [†]	Foam or NPWT	Foam or NPWT	Gelling Fibre or NPWT	NPWT and Skin Care
INTRASITE [®] GEL or INTRASITE CONFORMABLE	IODOFLEX [®] or IODOSORB [®] Range	ACTICOAT [®] Range	ALLEVYN [®] Range*	ACTICOAT Range	DURAFIBER [®] Ag	IODOFLEX or IODOSORB Range	ALLEVYN GENTLE BORDER LITE or PICO**	ALLEVYN GENTLE BORDER or PICO [‡]	ALLEVYN LIFE or RENASYS**	DURAFIBER or PICO [§] or RENASYS	PICO or RENASYS SECURA [®] / PROSHIELD [®] Range**

Wound management goal

Viable (vascularised) wound bed	Reduced bacterial burden and inflammation	Optimal moisture balance	Advancing edge of wound
---------------------------------	---	--------------------------	-------------------------

Evaluation of treatment and wound management goal.
Select dressing frequency or change treatment.

Reference: 1. Schultz GS, Sibbald RG, Falanga V et al., Wound Rep Reg (2003);11:1-28. 2. Leaper DJ, Schultz G, Carville K, Fletcher J, Swanson T, Drake R. Extending the TIME concept: what have we learned in the past 10 years? Int Wound J 2012; 9 (Suppl. 2):1-19. *ALLEVYN Range includes ALLEVYN LIFE, ALLEVYN GENTLE BORDER and ALLEVYN GENTLE BORDER LITE. [†]NPWT: Negative Pressure Wound Therapy. [‡]Level of exudate for wounds suitable for NPWT. [§]Wounds treated with the larger dressing sizes of the PICO system should generally be no more than 2cm (4/5 in.) in depth. **SECURA Range includes SECURA Moisturising Cleanser, SECURA Total Body Foam, SECURA Dimethicone Protectant, SECURA Extra Protective Cream, No Sting Skin Prep; PROSHIELD Range includes PROSHIELD Plus and PROSHIELD Foam & Spray.

Smith & Nephew does not provide medical advice. The information presented is not, and is not intended to serve as, medical advice. For detailed device information, including indications for use, contraindications, precautions and warnings, please consult the product's Instructions for Use (IFU) prior to use. It is the responsibility of healthcare professionals to determine and utilize the appropriate products and techniques according to their own clinical judgment for each of their patients.

Smith & Nephew Croxley Park, Building 5, Lakeside, Hatters Lane, Watford, Hertfordshire, WD18 8YE, UK. T +44 (0) 1923 477100 F +44 (0) 1923 477101
[®]Trademark of Smith & Nephew All Trademarks acknowledged. ©March 2018 Smith & Nephew. 12877